Vario 18

SUCTION PUMP

- EN Patient Instructions for use
- DE Gebrauchsanweisung für Patienten
- FR Mode d'emploi
- IT Istruzioni per l'uso per i pazienti
- NL Gebruiksaanwijzing voor patiënten
- SV Patientbruksanvisning
- FI Käyttöohjeet potilaalle
- NO Pasientbruksanvisning
- ES Instrucciones de uso para el paciente
- PT Instruções de utilização para o paciente

Vario 18 AC/DC

Assistance

Read the entire instruction manual before trying to operate the device.

If you have any questions about your Vario 18 suction pump, please contact your healthcare provider/physician at the number below:

HEALTHCARE PROVIDER/PHYSICIAN CONTACT
INFORMATION:

Keep this Patient Instructions for use booklet in an easily accessible location.

Table of contents

Warnings and safety instructions	4
Description of the device	7
Set up instructions.....	10
Preparation for use	11
Operating instructions.....	17
Filter test.....	21
Changing reusable jar	22
Changing disposable liner.....	24
Troubleshooting	26
Battery data and test.....	28
Battery calibration.....	30
Cleaning guidelines.....	31
Signs and symbols	33
Warranty and maintenance	35
Disposal	35
Technical specifications	36

Warnings and safety instructions

WARNINGS

The individual responsible for operating this device on a daily basis (patient, caregiver) must be able to read, understand and follow directions provided in these instructions for use, and by the healthcare professional. The individual must be able to hear and respond to auditory signal.

Only use the Vario 18 on the person for whom it was ordered and only for its intended use. Do not perform the therapy without your healthcare provider's supervision.

The Vario 18 is approved exclusively for the use as described in these Patient Instructions for use. Medela can only guarantee the safe functioning of the system when the Vario 18 is used in combination with the original Medela accessories.

Please read and observe these warning and safety instructions before operation. These Patient Instructions for use must be kept with the device for later reference.

Please note that these instructions for use are a general guide for the use of the product. Medical matters must be addressed by a physician.

- If you are dependent on the device for airway suctioning and a breakdown can lead to a critical situation, you must have a replacement device at hand.
- The connecting tubing delivered with the device must always be connected to a sterile catheter as prescribed by your physician.
- Do not modify the equipment.

General safety at home

- The use of mobile telephones, LAN/WLAN, walkie-talkies (two-way radios) and cordless telephones sets can affect the Vario 18 pump. A safety distance of min. 3.3 ft (1 m) to the Vario 18 pump is recommended.
- Keep the power supply cord away from hot surfaces.
- Keep the mains plug and the on/off switch away from moisture.
- Never pull the mains plug out of the fixed mains socket by pulling on the power supply cord!
- Never leave the device unattended when it is switched on.
- The pump must stand upright during use.
- Never use the device at high room temperatures, while bathing or showering or if you are very tired.

- Do not use extension cords with Vario 18 pump. Keep the power supply cord where you will not fall or trip over it.
- Never place the power supply cord around your neck.
- Keep the Vario 18 pump clean and dry.
- Never place the pump in water or liquids.
- If the pump gets wet, rub with dry towel. Do not dry in a microwave.
- Keep the pump away from children and pets.
- Keep the pump protected from direct sunlight.
- Prevent the pump from falling to the floor during sleep.
- Do not touch the pump when it has fallen into water. Unplug device immediately from electrical outlet.

When you leave the house

- Make sure the LED indicator is not blinking when you leave the house.
- For your convenience put the pump into the optional carrying bag and store the tubing safely in the tubing bag.
- Don't forget to take a spare jar/liner/tubing and the power supply with you.

These instructions for use must be kept for later reference.

Description of the device

Introduction

Your physician has prescribed the Vario 18 for airway suctioning. This suction pump from Medela is a high-quality suction pump that combines easy handling and cleaning with safety features to ensure optimal operation.

Intended use

The Vario 18 suction pump is indicated to clear the airways either through the nose, mouth or tracheostomy tube. The connecting tubing delivered with the device must always be connected to the catheter prescribed by your physician.

Intended user

The Vario 18 should only be operated by properly trained persons.

Service life

The service life of the device is three years.

Caution

Federal US law restricts this device to sale by or on the order of a health professional.

Subject to change.

Versions of the pump

Vario 18 AC


Vario 18 AC/DC

Front of the pump

Operating elements and battery status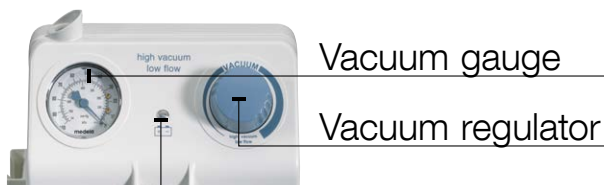

	Only AC/DC version:
not lit	Battery is fully charged
lit up	– Battery is being charged – Pump is connected to the mains
blinks slowly	Battery is soon empty
blinks rapidly	Battery is defective

Acoustic indicator

During battery operation an acoustic signal (beep - beep - beep) indicates that the rechargeable battery is soon discharged.

On/off switch

On: pump is running
Off: pump is switched off

Set up instructions

Check initial delivery

Check the delivery package of the Vario 18 for completeness and general condition.

1 x Vario 18 suction pump

1 x Silicone Tubing with 2 coupling pieces

5 x Disposable filters

1 x Instructions for use

Accessories

Please ask your healthcare provider / physician in case of missing parts or for additional accessories.

Preparation for use

WARNINGS

For use only by persons who have been adequately trained in the use of suction pumps for airway suctioning by their healthcare provider/physician.

CAUTIONS

- The Vario 18 must remain in an upright position during use.
- Sterile accessories must be checked on the integrity of the packaging before use.
- Non sterile and reusable accessories must be cleaned and disinfected before use.

Checks before use

- Check the Vario 18 pump before use for damage of the power cord or plug, obvious device damage or safety defects and proper functioning of the device.
- Only for AC/DC versions: make sure that the rechargeable battery is calibrated according to chapter «battery calibration».
- Check all accessories prior to use:
 - suction jars, lids and liners for cracks, brittle and flawed spots. Replace if necessary.
 - Tubing for cracks, brittle areas and that connectors are firmly attached. Replace if necessary.

Basic assembly

1.  Attach the filter to the Vario 18 with the arrow pointing towards the pump. The filter must be exchanged on a weekly basis.

WARNINGS

The filter protects the pump against overflow.

When the filter is wet/moist/clogged you will not be able to suction. Test the filter before each use (see chapter «filter test»).

2. Attach all necessary accessories according to your healthcare provider's instructions.

Reusable collection system

1. If you use a disposable collection system proceed with chapter «disposable collection system».

a) Attach floater to lid.

b) Make sure the floater moves up/ down freely.

a) Attach the lid to the jar.

b) Lock with the two lid clamps.

a) Attach the jar to the pump.

b) Connect tubing to the filter.

c) Connect tubing to the lid of the jar.
The port is marked with «vacuum».

5.  Connect patient tubing (the fingertip tubing delivered with the pump) to the lid of the jar. The port is marked with «patient».

CAUTION

Do not use tubing or other sterile accessories if the sterile packaging is damaged. Do not reuse single use or sterile products.

Disposable collection system (when using liners)

1. If you use a reusable collection system proceed with chapter «reusable collection system».

2. a) Attach the Vario 18 jar adapter (blue piece) to the pump's jar holder.

3. a) Prepare the 1.5l jar (yellow stripe).
b) Insert the liner with the yellow plugs on top according to instructions delivered with liners.

4. a) Attach the jar to the pump.
b) Connect tubing to the filter.
c) Connect tubing to the jar. The tubing port is on top of the jar.

5.

- a) Take the coloured angle piece from the lid of the liner and insert into the port labeled «patient».
- b) Connect patient tubing (the finger-tip tubing delivered with the pump) to the coloured angle piece.

CAUTION

Do not use tubing or other sterile accessories if the sterile packaging is damaged. Do not reuse single use or sterile products.

Operating instructions

CAUTIONS

The Vario 18 is to be set up in such a way, that it can easily be unplugged from the mains socket.

Connect Vario 18 to mains power

1. Check the pump before use following the instruction in chapter «preparation for use».

2. **All versions:** Plug in the mains plug of the power cord to a fixed mains socket.

or

AC/DC

version: (if the pump is operated using the optional car connection cable)

Connect the optional 12 V DC connection cable to the socket on the side of the pump and connect it to the 12 V DC power source (cigarette lighter in the car).

or

AC/DC

version: (if the pump is operated with the rechargeable battery)

No connection to a power source is needed. Make sure that the battery was charged for five hours.

Check maximal vacuum for functional check

1.  Switch on the Vario 18. The pump will start running.

2.  Press and turn the vacuum regulator to the right to set maximum vacuum.

3. 
 - a) Seal the end of the patient tubing with your thumb.
 - b) Compare the maximum vacuum according to the specification.
If the maximum vacuum is not reached, see chapter «trouble-shooting».

Specifications:

Altitude above

sea level:

Max. Vacuum:

▼	▼
+ 2000 m	– 59 kPa
+ 1000 m	– 66 kPa
+ 500 m	– 70 kPa
+ 0 m	– 75 kPa

(Tolerance +/- 10%)

Changing vacuum level

Use only vacuum level recommended by your healthcare provider / physician.

1.

a) Clamp patient tubing

b) Push and turn vacuum regulator to select the correct vacuum according to the instructions from your physician.

c) Check vacuum gauge for setting.

Placing out of operation after use

1.  Switch the off switch of the Vario 18 to position «0».

2. **If the pump is connected to a fixed mains socket:** Disconnect the mains plug from the fixed mains socket.

or

- If the pump is connected to a 12VDC power source:** Disconnect the 12VDC plug from the 12VDC power source.

3. Clean and disinfect the Vario 18. See chapter «cleaning guidelines».

Filter test

1.

Remove tubing from filter.

2.

Select maximum vacuum.

3.

Switch on the Vario 18. The pump will start running.

4.

a) Read the vacuum.

b) Replace the filter if the vacuum exceeds 10kPa.

Changing reusable jar

1.

a) Switch the Vario 18 off.

b) Remove patient and vacuum tubings from the lid.

2.

Remove full jar from the Vario 18.

3.

a) Prepare new jar and unpack it.

b) Attach new jar to the Vario 18.

c) Reconnect vacuum tubing and new patient tubing firmly.

4. Empty jar and dispose of patient tubing according to the instructions from your healthcare provider.
5. Switch on the Vario 18. The pump will start running.
6. Set vacuum according to the instructions from your physician. See chapter «changing vacuum level».


Changing disposable liner

WARNINGS

Do not reuse disposable liners.

1.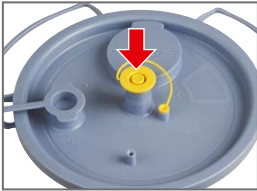

- a) Remove patient tubing with colored angle piece from lid.
- b) Plug patient port.

2.

Switch the Vario 18 off.

Reduce vacuum to minimum by pressing and turning the vacuum regulator to the left.

3.

Remove full liner from the jar.

4. Dispose of the liner and patient tubing according to the instructions from your healthcare provider.

5. Prepare new liner, unpack it and insert it in the jar.
Press down lid.

6. Connect new patient tubing firmly.

7. Switch on the Vario 18. The pump will start running.

8. Set vacuum according to the instructions from your physician. See chapter «changing vacuum level».

Troubleshooting

Motor not running

Check if:

- the Vario 18 is switched on.
- the mains voltage is correct and if the mains plug is inserted correctly into the fixed mains socket.
- the internal battery is charged (only AC/DC version under battery operation)

Suction of pump too low

Check if:

- the vacuum regulator is set correctly.
- the tubings are not defective or broken. If necessary, replace.
- all plug-in connections are tight.
- the floater on the reusable lid moves freely up/down.
- the suction jar/lid has no cracks, brittle areas, discolouration. Replace if necessary.
- the disposable system has no cracks, brittle areas, discolouration. Replace if necessary.
- filter is not clogged. To test if the filter is clogged see chapter «filter test».

AC/DC version only:

No LED lit

The rechargeable battery is fully charged.

LED lit up

- The rechargeable battery is being charged.
- The pump is connected to the mains.

LED blinks slowly

The rechargeable battery is soon empty.

LED blinks rapidly

The rechargeable battery is defective.

CAUTIONS

Do not try to fix or repair the Vario 18 pump. Contact your healthcare provider immediately if:

- if an overflow into the device has occurred
- if the power cord or the plug are damaged
- if the device is not functioning perfectly
- if the device is damaged
- if the device shows clear safety defects.

If the fault cannot be rectified, please contact your healthcare provider.

Battery data and test

(AC/DC version only)

CAUTIONS

Only charge the rechargeable battery when the acoustic signal sounds and the charging indicator blinks slowly. The rechargeable battery is charged during mains operation.

The Vario 18 is a mains- and battery powered suction pump. One full battery charge (Vario 18 AC/DC version) would last you a total of 30 minutes continuous running time. Then the battery must be recharged.

If you are not using your Vario 18 AC/DC version or storing for a longer period of time (> 2 months), recharge the battery every 60–90 days.

Battery low

Only recharge the battery when the LED indicator blinks slowly and you hear the acoustic signal (beep - beep - beep). You have 10 minutes of suctioning time left.

Battery empty

As soon as the rechargeable battery is empty, the pump comes to a stop. The rechargeable battery must be recharged.

Duration of pumping

If the battery is fully charged, the operating time is about 30 minutes.

Charging time

Charged up to 100 % in approx. 5 hours.

Testing the rechargeable battery

1. Make sure that the rechargeable battery is fully charged.
2. Pull out the mains plug from the fixed wall outlet.

3.  Press vacuum regulator and turn it to the right to select maximum vacuum.

4.  Switch on the Vario 18. The pump will start running.

5. Measure the time until the pump comes to a stop.
The battery is ok, when pump runs for more than 30 minutes.

If the values are not reached, repeat test or calibrate the battery (see next page).

Battery calibration

(AC/DC version only)

CAUTIONS

Calibrate rechargeable batteries before the first use, or after a storage period > 2 months. If the calibration process fails, return to your healthcare provider to have battery replaced.

Calibrating NiMH rechargeable battery

1. Make sure that the battery is fully charged.
2. Pull out the mains plug from the fixed wall outlet.
3.  Switch on the Vario 18. The pump will start running.

4. Let the pump run until the rechargeable battery is empty and the pump comes to a stop.
5. Plug the mains cable into the wall outlet.
The rechargeable battery is being charged.
6. Wait until the charging indicator LED goes out.
7. Repeat steps 1 to 6. The Vario 18 AC/DC is then ready to use.

Storage of the rechargeable battery

In order to keep the self-discharge at a minimum, store the Vario 18 and the replacement rechargeable batteries at temperatures below 25 °C (77 °F). Repeat the calibrating process every 60–90 days.

Cleaning guidelines

WARNINGS

Before cleaning the device, pull the mains plug out of the fixed mains socket.

General notes

- Follow the cleaning instructions given by your healthcare provider
- Wear protective gloves for cleaning / disinfection.
- Dispose of fluids such as blood and secretions and the parts contaminated with them according to your healthcare provider's instructions.

Water

Use only the purest quality of water for cleaning. Water hardness is a serious consideration since deposits left on medical products may not be properly decontaminated. Use deionised water in order to reduce this problem.

Disposable products

② These are single use products not intended to be reused. Reuse could cause loss of mechanical, chemical and/or biological characteristics. Reuse could cause cross contamination.

Disassembly

Separate all individual parts before cleaning and disinfecting.

Cleaning jars and lids

Clean components in hot water (60–70 °C) containing a detergent with a pH range between 6.0 and 8.0 only.

1. Soak all parts thoroughly with warm, soapy water (60–70 °C) or in enzymatic detergent for 1–5 minutes.
2. Remove visible dirt with a cleaning tool – general purpose cleaning brushes, such as pipe cleaners or non-abrasive lint cloths. Rinse thoroughly in clear water. Allow to dry.
3. Check the parts for visible dirt and repeat these steps if necessary.

Alternatively the jars/lids can be placed in a dishwasher for cleaning.

Cleaning pump housing, mains cable and plastic accessories

Wipe with detergent recommended by your healthcare provider. Use clean cloth to dry.

Signs and symbols

0123 This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.

This symbol indicates the legal specifications of the pump.

This symbol indicates to follow instructions for use.

This symbol indicates to consult instructions for use.

This symbol indicates a CAUTION or WARNING associated with the device.

This symbol indicates safety related tip.

This symbol indicates the class of the pump.

This symbol indicates a class II device.

This Symbol indicates the protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.

This symbol indicates a type CF applied part.

This symbol indicates MR UNSAFE.

This symbol indicates that interferences may occur in the vicinity of equipment marked with this symbol.

This symbol indicates the manufacturer.

This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

This symbol indicates that the device should not be used after the end of the year and month shown.

This symbol indicates a single use device. Do not reuse the device.

This symbol indicates manufacturer's catalogue number.

This symbol indicates manufacturer's serial number.

This symbol indicates manufacturer's batch code.

This symbol indicates the device is sterilized using ethylene oxide.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This Symbol indicates a Prescription Device. Federal Law restricts this device to sale by or on the order of a licensed healthcare practitioner. (for US only).

This symbol indicates do not use the device if package is damaged.

This symbol indicates the maximum vacuum level of the pump.

This symbol indicates number of items.

This symbol indicates the number of items n that the content is sufficient for.

This symbol indicates the flow levels of the pump.

This symbol indicates the fuse

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates the electrical specifications of the pump.

This symbol indicates alternating current.

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates direct current.

This symbol indicates a carton package.

This symbol indicates the weight of the pump.

This symbol indicates to keep the device away from sunlight.

This symbol indicates the dimensions (h x w x d) of the pump.

This symbol indicates to handle the fragile device with care.

Warranty and maintenance

Warranty

Warranty period is for 2 years (6 months for the rechargeable battery) after date of delivery. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons.

Maintenance

The Vario 18 Pump should not require maintenance. If a Vario 18 pump fails within the warranty period due to a manufacturing defect, it will be replaced. The original pump will need to be returned to the supplier.

Disposal

Ask your healthcare provider how to dispose of used tubings/liners.

Technical specifications

CAUTIONS

Do not operate the Vario 18 Pump above 9,842 feet above sea level (3000m above sea level).

Storage conditions

The Vario pump and accessories must remain in the packaging for storage and stored at a temperature range of -13°F and $+158^{\circ}\text{F}$ (-25°C and $+70^{\circ}\text{C}$).

Rechargeable battery $+32^{\circ}\text{F}$ and $+77^{\circ}\text{F}$ (0°C and $+25^{\circ}\text{C}$)

Operation temperature

The Vario 18 Pump and accessories must be operated within a temperature range of $+41^{\circ}\text{F}$ and $+104^{\circ}\text{F}$ ($+5^{\circ}\text{C}$ and $+40^{\circ}\text{C}$). Do not operate the products in extreme cold or heat. Avoid high humidity (15–93 % is permissible).

Protection class IP21

The Vario 18 Pump is protected against ingress of dripping water.

high vacuum

– 75 kPa / – 563 mmHg (Tolerance: – 10 %)

Measured at 0 m, atmospheric pressure:
1013.25 hPa. Please note: vacuum levels may
vary depending on location (meters above sea
level, atmospheric pressure and temperature).

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Medela AG
Medela UK Ltd.
Medela Inc.